



Your PDF Guides


You can read the recommendations in the user guide, the technical guide or the installation guide for SENNHEISER IS 410. You'll find the answers to all your questions on the SENNHEISER IS 410 in the user manual (information, specifications, safety advice, size, accessories, etc.). Detailed instructions for use are in the User's Guide.

User manual SENNHEISER IS 410
User guide SENNHEISER IS 410
Operating instructions SENNHEISER IS 410
Instructions for use SENNHEISER IS 410
Instruction manual SENNHEISER IS 410

 **SENNHEISER**

IS 410

Instructions for use

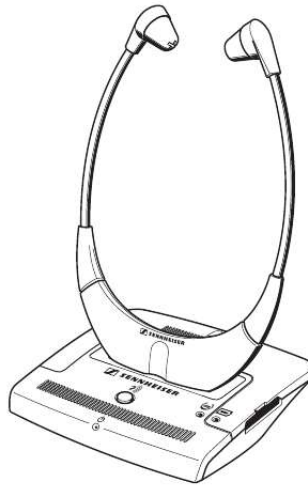

[You're reading an excerpt. Click here to read official SENNHEISER IS 410 user guide](http://yourpdfguides.com/dref/3870065)

<http://yourpdfguides.com/dref/3870065>

Manual abstract:

.....
.....
.....
.....

.....
.....
.....
.....

.... 2 The IS 410 stethoset receiver system .

.....
.....

.....
.....
.....
.....

3 Delivery includes

.....
.....
.....

.....
.....
.....
.....

.....
.....

... 3 Operating controls ..

.....

.....
.....
.....
.....

.....
.....
.....
.....

.... 4 Receiver (see inside cover)

.....
.....
.....

.....
.....
.....
.....

. 4 Transmitter (see inside cover)

.....
.....

.....
.....
.....
.....

.....
.. 4 LED indications (see inside cover) ...

.....
.....

.....
.....
.....
.....

..... 4 Connection possibilities (see inside cover) ...

.....
.....
.....

.....
.....

. 4 Putting the system into operation

.....
.....
.....
.....
.....

..... 5 Setting up the transmitter

.....
.....
.....

.....
.....
.....
.....

.....
... @@@@8 Using the system components ..

.....
.....
.....
.....
.....
.....

..... 9 Switching the transmitter on .

.....
.....
.....
.....
.....
.....

.....
.. 9 Switching the stehoset receiver on ...

.....
.....
.....
.....
.....

..... 9 Adjusting the volume

.....

.....
.....

.....
.....
.....

.....
.....

.... 10 Adjusting the balance .

.....

.....
.....
.....

.....
.....
.....

... 10 Enhancing speech intelligibility

.....
.....
.....

.....
.....
.....

10 Storing and charging the receiver

.....
.....
.....

.....
.....
.....

... 11 Meaning of the LEDs and acoustic signals

.....
.....
.....

..... 12 LEDs on the transmitter .

.....
.....
.....

.....
.....
.....

..... 12 Acoustic signals of the receiver .

.....
.....
.....

.....
.....
.....

..... 12 Care and maintenance

.....
.....
.....
.....
.....
.....
.....

.... 13 Cleaning the system .

.....
.....
.....
.....
.....
.....
.....
.....

. 13 Cleaning the ear cushions

.....
.....
.....
.....
.....
.....

..... 13 If problems occur..

.....
.....
.....
.....
.....
.....
.....
.....

.... 14 Accessories

.....
.....
.....
.....
.....
.....
.....
.....

15 Connector assignment

.....

Is the green operation indication lit? Is the volume of the receiver sufficient? Is the rechargeable battery fully charged and inserted properly into the receiver? Do you wear the receiver the right way round, with the Sennheiser logo at the front? Is there a free line of sight between transmitter and receiver? The speech intelligibility is reduced You hear the sound only with one ear Is the "Speech intelligibility" button backlit in yellow? Is the balance adjusted correctly? Page 5 ff 5 ff 7 6 7 10 8 9 5 10 10 If problems occur that are not listed in the above table or if the problems cannot be resolved with the proposed solutions, please contact your local Sennheiser agent for assistance. 14 Accessories Attention! Components from other manufacturers (e.g. for power supply or rechargeable battery) may cause damage to the unit and will invalidate the warranty.

Only use original Sennheiser accessories and spare parts. The following accessories are available from your authorized Sennheiser dealer: BA 300 spare rechargeable battery Cat. No. 500898 RI 410 receiver Cat. No.

500684 Ear cushions (normal), 5 pairs Cat. No. 517625 Ear cushions (lamellar form), 5 pairs Cat. No. 517626 SCART adapter Cat. No. 077798 3 m extension cable Cat. No. 517627 (3.5 mm stereo jack plug) MKE 800 TV-N Cat.

No. 500974 external microphone (mono) Connector assignment The blue socket AUDIO is designed for use with two different connector types (Line/Mic).



[You're reading an excerpt. Click here to read official SENNHEISER IS 410 user guide](http://yourpdfguides.com/dref/3870065)

<http://yourpdfguides.com/dref/3870065>

The connector type used is identified automatically. 3.5 mm 3-pole jack/standard stereo L R GND 3.5 mm 4-pole jack/microphone with additional power supply L R 8V GND 1k 15 Specifications System Modulation Carrier frequencies Range AF frequency response THD FM stereo 2.3 / 2.8 MHz approx 12 m within a room 50 Hz 16 kHz <1% Transmitter Audio /microphone connection Charging time of rechargeable battery Power supply Weight Dimensions 3.5 mm stereo jack socket approx. 3 hrs 12 VDC/50 mA via NT 410 mains unit approx.

140 g approx. 115 x 145 x 40 mm Receiver Max. SPL Power supply Operating time Weight Dimensions 125 dB (SPL) via BA 300 Li-Ion rechargeable battery (3.7 V / 150 mAh) approx. 16 hrs approx.

50 g approx. 230 x 135 x 20 mm 16 Manufacturer Declarations Warranty regulations The guarantee period for this Sennheiser product is 24 months from the date of purchase. Excluded are accessory items, rechargeable or disposable batteries that are delivered with the product; due to their characteristics these products have a shorter service life that is principally dependent on the individual frequency of use. The guarantee period starts from the date of original purchase. For this reason, we recommend that the sales receipt be retained as proof of purchase.

Without this proof (which is checked by the responsible Sennheiser service partner) you will not be reimbursed for any repairs that are carried out. Depending on our choice, guarantee service comprises, free of charge, the removal of material and manufacturing defects through repair or replacement of either individual parts or the entire device. Inappropriate usage (e.g. operating faults, mechanical damages, incorrect operating voltage), wear and tear, force majeure and defects which were known at the time of purchase are excluded from guarantee claims. The guarantee is void if the product is manipulated by non-authorized persons or repair stations. In the case of a claim under the terms of this guarantee, send the device, including accessories and sales receipt, to the responsible service partner. To minimise the risk of transport damage, we recommend that the original packaging is used. Your legal rights against the seller, resulting from the contract of sale, are not affected by this guarantee. The guarantee can be claimed in all countries outside the U.

S. @@@@In order to protect the environment, only dispose of exhausted batteries. WEEE Declaration Your Sennheiser product was developed and manufactured with high-quality materials and components which can be recycled and/or reused. This symbol indicates that electrical and electronic equipment must be disposed of separately from normal waste at the end of its operational lifetime. Please dispose of this product by bringing it to your local collection point or recycling centre for such equipment. This will help to protect the environment in which we all live. 17 Sennheiser electronic GmbH & Co. KG 30900 Wedemark, Germany Phone +49 (5130) 600 0 Fax +49 (5130) 600 300 www.sennheiser.com Printed in China Publ. 10/06 517637 / A02 .



[You're reading an excerpt. Click here to read official SENNHEISER IS 410 user guide](#)
<http://yourpdfguides.com/dref/3870065>