



# Your PDF Guides


You can read the recommendations in the user guide, the technical guide or the installation guide for SENNHEISER SET 50 TV. You'll find the answers to all your questions on the SENNHEISER SET 50 TV in the user manual (information, specifications, safety advice, size, accessories, etc.). Detailed instructions for use are in the User's Guide.

**User manual SENNHEISER SET 50 TV**  
**User guide SENNHEISER SET 50 TV**  
**Operating instructions SENNHEISER SET 50 TV**  
**Instructions for use SENNHEISER SET 50 TV**  
**Instruction manual SENNHEISER SET 50 TV**

 **SENNHEISER**

Set 55 TV  
Set 50 TV

Instructions for use

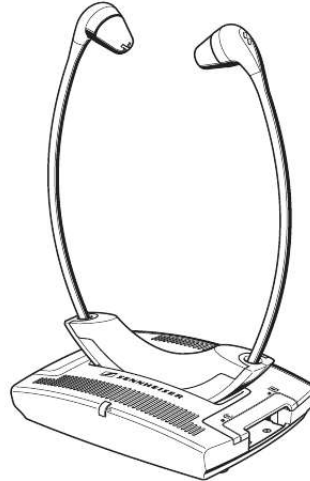

[You're reading an excerpt. Click here to read official SENNHEISER SET 50 TV user guide](http://yourpdfguides.com/dref/3706753)  
<http://yourpdfguides.com/dref/3706753>

**Manual abstract:**

.....  
.....  
.....  
.....

.....  
.....  
.....  
.....

.....  
... 2 *The Set 50 TV/Set 55 TV stethoset receiver system* ..

.....  
.....

..... 3 *Set 50 TV: Delivery includes* .....

.....  
.....  
.....

.....  
.....  
.....  
.....

..... 3 *Set 55 TV product variant* .....

.....  
.....  
.....

.....  
.....  
.....  
.....

.....  
... 4 *Operating controls* ..

.....  
.....

.....  
.....  
.....  
.....

.....  
.....  
.....

..... 4 *Receiver (see inside cover)* ..

.....  
.....  
.....  
.....

.....  
.....  
.....  
.....

.... 4 *Transmitter (see inside cover)* .....

.....  
.....  
.....

.....  
.....  
.....  
.....

... 4 LED indications (see inside cover) .....

.....  
.....  
.....

.....  
.....  
.....

..... 4 Connection possibilities (see inside cover) ..

.....  
.....  
.....  
.....

..... 4 Putting the system into operation ....

.....  
.....  
.....

.....  
.....  
.....

..... @@@@8 Inserting a spare rechargeable battery into the transmitter's charging compartment .....

.....  
.....  
.....  
.....

... 8 Using the system components ..

.....  
.....  
.....

.....  
.....  
.....

..... 9 Switching the transmitter on ..

.....  
.....  
.....

.....  
.....  
.....

9 Switching the receiver on .....

.....  
.....  
.....

.....  
.....  
.....

.. 9 Adjusting the volume .....

.....

.....  
.....

.....  
.....  
.....

.....  
.....

*.. 9 Adjusting the balance ...*

.....

.....  
.....  
.....

.....  
.....  
.....

*10 Storing and charging the receiver .....*

.....  
.....  
.....

.....  
.....  
.....

*.. 10 Meaning of the LEDs .....*

.....  
.....  
.....

.....  
.....  
.....

*.. 11 LEDs on the transmitter .....*

.....  
.....  
.....

.....  
.....  
.....

*. 11 Care and maintenance ....*

.....  
.....

.....  
.....  
.....

.....  
.....

*... 12 Cleaning the system ..*

.....

.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....

... 12 Cleaning the ear cushions .....

.....  
.....  
.....  
.....  
.....  
.....

12 If problems occur... ..

.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....

..... 13 Accessories .....

.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....  
.....

14 Connector assignment .....

.....  
.....  
.....  
.....  
.....  
.....  
.....

..... 14 Specifications .....

.....  
.....  
.....  
.....  
.....

.....  
.....  
.....

.. 15 Manufacturer Declarations ...

.....  
.....  
.....  
.....  
.....  
.....

16 Thank you for choosing Sennheiser! We have designed this product to give you reliable operation over many years. Over 60 years of accumulated expertise in the design and manufacture of high-quality electro-acoustic equipment have made Sennheiser a world-leading company in this field. Please take a few moments to read these instructions carefully, as we want you to enjoy your new Sennheiser products quickly and to the fullest. 1 Safety instructions Please read these instructions carefully and completely before using the system.

Make these instructions easily accessible to all users at all times. Always include these instructions when passing the system on to third parties. Do not wear the headphones in situations which require special attention (e.g. when performing skilled jobs). Always keep the system dry and do not expose it to extreme temperatures (ideally 10-35°C). Use the system with care and set it up in a clean, dust-free environment. Varnish or furniture polish may degrade the rubber feet of the transmitter, which could stain your furniture. You should therefore place the transmitter on a non-slip pad to avoid potential staining of furniture. This system allows you to set higher volumes than conventional stethoset systems.

This can cause hearing damage in persons with healthy hearing and especially in children. Please observe the safety instructions for NiMH rechargeable batteries on the enclosed leaflet. The volume can be adjusted directly on the receiver. Note: Avoid exposure to direct sunlight. Check the connection possibilities of your sound source (e.g. This improves the quality of cordless sound transmission. If necessary, check the menu setting of your TV. Depending on the socket, you may have to use the adapter ! Connect the connecting cable to the SCART adapter.

Plug in the SCART adapter to the SCART socket on your TV. Plug the mains connector into a wall socket. The LED lights up green. If the receiver is stored in the transmitter's charging compartment, the LED goes off. Note: Pull the mains connector out of the wall socket when the transmitter will not be used for extended periods of time (e.g. when you are on holiday). In this case, however, the rechargeable battery will not be recharged. Inserting and removing the receiver's rechargeable battery Insert the battery into the compartment as shown on the left. Make sure that the battery is inserted correctly.

To remove the battery, use the finger catch projection to pull the battery out of the receiver. Note: The receiver is delivered with the battery inserted. Charge the battery for at least 3 hours before you use it for the first time (see "Storing and charging the receiver" on page 10). Inserting a spare rechargeable battery into the transmitter's charging compartment You can charge a spare battery (not included in the delivery of the Set 50 TV) in the charging compartment at the side of the transmitter. This spare battery is not required for operating the Set 50 TV/Set 55 TV but offers extended listening time while the discharged first battery is being recharged. Insert the spare battery into the charging compartment as shown on the left. If the rechargeable battery is inserted correctly, the LED lights up red. The red LED does not go off when the battery is fully charged. 8 Using the system components Switching the transmitter on Take the stethoset receiver out of the transmitter's charging compartment. The transmitter automatically switches on.

The LED lights up green. Note: If the receiver is placed into the transmitter's charging compartment, the transmitter automatically switches off. The LED goes off. Switching the stethoset receiver on Attention! Volume up? - No! When people use headphones, they tend to choose a higher volume than with loudspeakers. Listening at high volume levels for long periods can lead to permanent hearing defects. Please protect your hearing, Sennheiser headphones also sound exceptionally good at low volumes. Turn the volume control it clicks. in the direction of the arrow until Before putting the receiver on, use the volume control to adjust the volume to a low level. This protects your hearing from damage. 3 Put the receiver on the correct way round, with the Sennheiser logo at thOnly use original Sennheiser accessories and spare parts.

The following accessories are available from your authorized Sennheiser dealer: BA 151 spare rechargeable battery Cat. No. 004146 Ear cushions (normal), 5 pairs Cat. No. 517625 Ear cushions (lamellar form), 5 pairs Cat. No. 517626 SCART adapter Cat. No. 077798 3 m extension cable Cat. No.

517627 (3.5 mm stereo jack plug) MKE 800 TV-N Cat. No. 500974 external microphone (mono) Connector assignment The blue socket AUDIO INPUT is designed for use with two different connector types (Line/Mic). The connector type used is identified automatically. 3.5 mm 3-pole jack/standard stereo L R GND 3.5 mm 4-pole jack/microphone with additional power supply 14 L R 8V GND 1k Specifications System Modulation Carrier frequencies Range AF frequency response THD FM mono 2.3 MHz approx 12 m within a room 50 Hz/12 kHz <1% Transmitter Audio /microphone connection Charging time of rechargeable battery Power supply Weight Dimensions 3.5 mm stereo jack socket approx.

14 hours 12 VDC/200 mA via NT 5 mains unit approx. 90 g approx. 95 x 130 x 30 mm Receiver Max. SPL Power supply Operating time Weight Dimensions 120 dB (SPL) via BA 151 NiMH rechargeable battery (2.4 V / 80 mAh) approx. 9 hours approx. 55 g approx. 200 x 150 x 20 mm 15 Manufacturer Declarations Warranty regulations The guarantee period for this Sennheiser product is 24 months from the date of purchase. Excluded are accessory items, rechargeable batteries that are delivered with the product; due to their characteristics these products have a shorter service life that is principally dependent on the individual frequency of use. The guarantee period starts from the date of original purchase.

[You're reading an excerpt. Click here to read official SENNHEISER](#)



[SET 50 TV user guide](#)

<http://yourpdfguides.com/dref/3706753>

*For this reason, we recommend that the sales receipt be retained as proof of purchase. Without this proof (which is checked by the responsible Sennheiser service partner) you will not be reimbursed for any repairs that are carried out. Depending on our choice, guarantee service comprises, free of charge, the removal of material and manufacturing defects through repair or replacement of either individual parts or the entire device. Inappropriate usage (e.g. operating faults, mechanical damages, incorrect operating voltage), wear and tear, force majeure and defects which were known at the time of purchase are excluded from guarantee claims. The guarantee is void if the product is manipulated by non-authorized persons or repair stations. In the case of a claim under the terms of this guarantee, send the device, including accessories and sales receipt, to the responsible service partner. To minimise the risk of transport damage, we recommend that the original packaging is used. Your legal rights against the seller, resulting from the contract of sale, are not affected by this guarantee.*

*The guarantee can be claimed in all countries outside the U.S. @@@@In order to protect the environment, only dispose of exhausted rechargeable batteries. WEEE Declaration Your Sennheiser product was developed and manufactured with high-quality materials and components which can be recycled and/or reused. This symbol indicates that electrical and electronic equipment must be disposed of separately from normal waste at the end of its operational lifetime. Please dispose of this product by bringing it to your local collection point or recycling centre for such equipment. This will help to protect the environment in which we all live. 16 Sennheiser electronic GmbH & Co. KG 30900 Wedemark, Germany Phone +49 (5130) 600 0 Fax +49 (5130) 600 300  
www.sennheiser.*

*com Printed in China Publ. 10/06 517640 / A02 .*



[You're reading an excerpt. Click here to read official SENNHEISER SET 50 TV user guide](http://yourpdfguides.com/dref/3706753)  
<http://yourpdfguides.com/dref/3706753>