




Your PDF Guides

You can read the recommendations in the user guide, the technical guide or the installation guide for THOMPSON SURGICAL BILATERAL SYSTEM. You'll find the answers to all your questions on the THOMPSON SURGICAL BILATERAL SYSTEM in the user manual (information, specifications, safety advice, size, accessories, etc.). Detailed instructions for use are in the User's Guide.

User manual THOMPSON SURGICAL BILATERAL SYSTEM
User guide THOMPSON SURGICAL BILATERAL SYSTEM
Operating instructions THOMPSON SURGICAL BILATERAL SYSTEM
Instructions for use THOMPSON SURGICAL BILATERAL SYSTEM
Instruction manual THOMPSON SURGICAL BILATERAL SYSTEM

Thompson Retractor 

Bilateral System User Manual

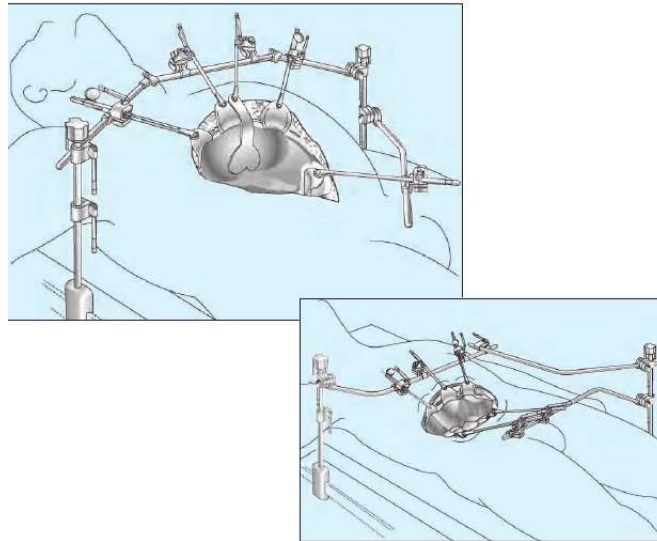

Uncompromised Exposure



[You're reading an excerpt. Click here to read official THOMPSON SURGICAL BILATERAL SYSTEM user guide](http://yourpdfguides.com/dref/3820458)
<http://yourpdfguides.com/dref/3820458>

Manual abstract:

*Elite II Rail Clamps Angled Arms *When necessary, use a wider OR table or add 2 1/2" to the width of the table by using our Rail Extender #41917. Angling Handles 4 Non-Angling Handles a CleAN All COmpONeNTs Do not allow blood to dry on components. Ultrasonic cleaning is recommended; however, hand washing with a brush is acceptable. lUbrICATe All ThreAded ANd Cam COmpONeNTs befOre AUTOClAvING Be certain that all Cam Joints are in full OpeN pOsITION before autoclaving. sTerIlIzE: GRAVITY STEAM (Wrapped) 30 minutes 121°C (250°F) @ 1.1kg/cm2 HIGH SPEED GRAVITY (FLASH) 5 minutes 132°C (270°F) @ 30psi or 132° @ 2kg/cm2 PREVACUUM STEAM (Wrapped) 10 minutes @132°C (270°F) CAUTION: relax retractors periodically to ensure proper blood flow. c b Tips a. Adjust angle of retractor blade before applying retraction. b. Compress the arm toward the incision while simultaneously retracting the blade.*

c. Have assistant lock joint. @@@@b. @@Add retractors: a. Press the plunger and insert blade into the handle.

b. Cam should be in "unlock" position c. Place blade in site d. Clip joint onto arm e. Retract and "lock" joint 5 a 6 a b b a.

Turn knob to fine tune retraction. b. @@Secure Extension Arms in the second joint on the Rail Clamps. Add retractors as needed. b. Secure additional Extension Arm if needed. @@@@b. @@@@b. Position the lower abdominal Crossbar in the Rail Clamp's lower Universal Joint and position it just above the patient's pubic bone. 3 b a 4 a.

Place the Lateral Arm in the top joint of the lower abdominal Rail Clamp. b. Interconnect its distal end with that of the upper abdominal's Crossbar, using 1/2" x 1/2" framing joint (42110C). Add retractors as needed. .



[You're reading an excerpt. Click here to read official THOMPSON SURGICAL BILATERAL SYSTEM user guide](http://yourpdfguides.com/dref/3820458)
<http://yourpdfguides.com/dref/3820458>